EN English

Introduction

You are purchasing a NC300 Mesh Nebulizer. With proper care and use-your nebulizer will provide you with many years of reliable treatment. This unit operates on standard AC power. Treatments are delivered quickly, safely and conveniently making this unit ideal for all ages. We encourage you to thoroughly read this guidebook to learn about the features of your nebulizer. Your mesh nebulizer should be used under the supervision of a physician or respiratory therapist. As a medical device, your physician and/or respiratory therapist, you can feel comfortable and confident knowing that you are obtaining the most effective inhalation treatments for your respiratory condition.

NOTE: Your nebulizer is intended for use in treatment of asthma, COPD and other respiratory ailments in which an aerosolized medication is required during therapy. Please consult with your physician and/or pharmacist to determine if your prescription medication is appropriate for use with this nebulizer. For type, dose, and regimen of medication follow the instructions provided by your physician.

This device fulfills the provision of the EC directive 93/42/EEC (Medical Device Directive) and the European Standard EN 13544-1:2007+A1:2009 Respiratory therapy equipment - Part 1: Nebulizing systems and their components. Please read this manual carefully before use and be sure to keep this manual.

Please read this manual carefully before use and be sure to keep this manual.

Cautions

Please use general safety precautions when operating your nebulizer. This unit should be used only for its intended purpose as described in this guidebook and with medications only under the supervision and instruction of your physician. Do not use the device in anesthetic or ventilator breathing circuits.

Operating your nebulizer

After every use:

1. Power off the device, unplug the device from the power source.

2. Carefully detach the accessories and medication cup from the nebulizer and pour out any remaining liquid.

3. Follow the cleaning procedures provided in this guidebook.

F Cleaning procedures

READ THE FOLLOWING BEFORE USING

To avoid electrical shock; keep unit away from water.

• Do not immerse the unit in liquid.

• Do not use while bathing.

• Do not re-use for a unit that has fallen into water immediately unplug the unit.

• Do not use for a unit that has any damaged parts (including pins). If it has been submerged in water or dropped, remove the unit for examination and repair.

• The unit should not be used where flammable gas, oxygen or aerosol spray products are being used.

• The medication cup is empty.

• If the medication cup has been disassembled, do not draw out the medication cup from the cup casing before the cup casing has been removed from the main unit so as to avoid the remaining liquid from penetrating inside the electrodes.

• Disconnect the unit from the electrical outlet before cleaning, filling and after each use.

• Do not use attachments unless recommended by the manufacturer.

• Do not plug in the device into an electrical outlet with wet hands.

• Do not use the device near an electrical outlet after using the device.

• Do not disassemble or attempt to repair the unit.

• Do not use the device in anesthetic or ventilator breathing circuits.

Operating cautions

• Close adult supervision - highly recommended when the unit is used by children or invalids.

• Children's eyes away from the output of medication mist.

• The maximum capacity of the medication cup is 8 ml and should not be overfilled.

• Do not use this unit while operating a vehicle.

• If any discomfort or abnormality occurs, stop using the unit immediately.

• Pentamidine is not an approved medication for use with this device.

• Do not use the device in anesthetic or ventilator breathing circuits.

• Do not use the device in anesthetic or ventilator breathing circuits.

• Do not touch the mesh with any towel or other material.

• Ensure the electrodes on medication cup fully dry and the medication cup is completely clean as based on the above cleaning process.

• The medication cup and Nebulizer unit life time will be shortened.

NOTE: For the first time cleaning or after the unit has been stored for a long period of time, thoroughly clean all components before using.

• If any discomfort or abnormality occurs, stop using the unit immediately.

• Pentamidine is not an approved medication for use with this device.

• Do not use the device in anesthetic or ventilator breathing circuits.

• Do not use the device in anesthetic or ventilator breathing circuits.

• Do not store the unit in direct sunlight, high temperature or humidity.

• Keep the unit out of reach of small children.

• Always keep the unit unplugged while not in use.

Cleaning cautions

• Check the mesh, mouthpiece, mask connector and any other optional component before use.

• Do not immerse the unit in water. It may damage the unit.

• Disconnect the unit from the electrical outlet before cleaning.

• Clean all necessary parts after each use as instructed in this guidebook.

• Always dispose of any remaining medication in the medication cup after each use. Use the medication cup for each unit you use the device.

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